# Field safety notice regarding *Shark* power module for Powered Wheelchair REAL 6100 PLUS

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## Dear customer,

We write to you to inform you that a problem has been identified with certain Dynamic Controls' Shark Power Modules. The Power Module is the electronics unit located in the rear of the chassis of the REAL 6100 PLUS.

Shark Power Modules manufactured between January 2018 and August 2018 may be susceptible to a broken internal spring clip due to a variation in heat treatment and plating process used by a supplier during spring clip manufacture. This problem has been isolated to a single batch of spring clips used in Shark Power Modules manufactured within the stated period of time. At Mercado Medic AB we have traced this to our powered wheelchairs with serial numbers according to Appendix A, which all require further actions. Around 40% of these chairs, 91 chairs in total, will need replacement of the Power Module, depending on the serial number of the Shark Power Module.

A broken spring clip inside a Shark Power Module present two hazardous scenarios of potential significance that may pose a risk of harm to users:

1. Metal spring clip fragments land on electronics and cause a short circuit of control circuitry, resulting in unintended wheelchair movement.
2. Metal spring clip fragments land on electronics and result in short circuit of power conductors, resulting in Shark Power Modules overheating due to continued flow of current from the fitted batteries.

Dynamic Controls and Mercado Medic has estimated that the likelihood of the specific sequence of events leading to a hazardous situation is very low. However, since the possibility of a hazardous scenario exists, the overall residual risk for the Shark Power Module has been deemed unacceptable.

Kindly follow the instructions in this document to (Step 1) confirm whether your chairs within the range of serial numbers in Appendix A are fitted with a defected Shark Power Module or not, and (Step 2) replace the Power Module if its serial number is on the list on page 5. Mercado Medic will provide you with free replacements where needed. Since the availability of the Shark Power Modules is limited we kindly ask for your understanding regarding that we can't send out more Power Modules than needed. Hence we ask you to identify the serial number (Step 1), before requesting replacements (Step 2). We want to emphasize the importance of you contacting your Mercado Medic representative as soon as you have identified an affected Power Module. Especially with regards to patient safety and the right to a replacement, but also to limit the amount of work involved for all parties in this matter. Please note also, that all Power Modules replaced need to be sent back to your Mercado Medic representative to be returned to Dynamic Controls.

Your Mercado Medic representative is available to provide additional technical detail and to support you through the replacement and resolution process.

Yours sincerely,

**Martin Bonnevier**

**Product development and Program Manager  
Responsible for Regulatory Compliance  
Mercado Medic AB**

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# Part 1: Check the serial number

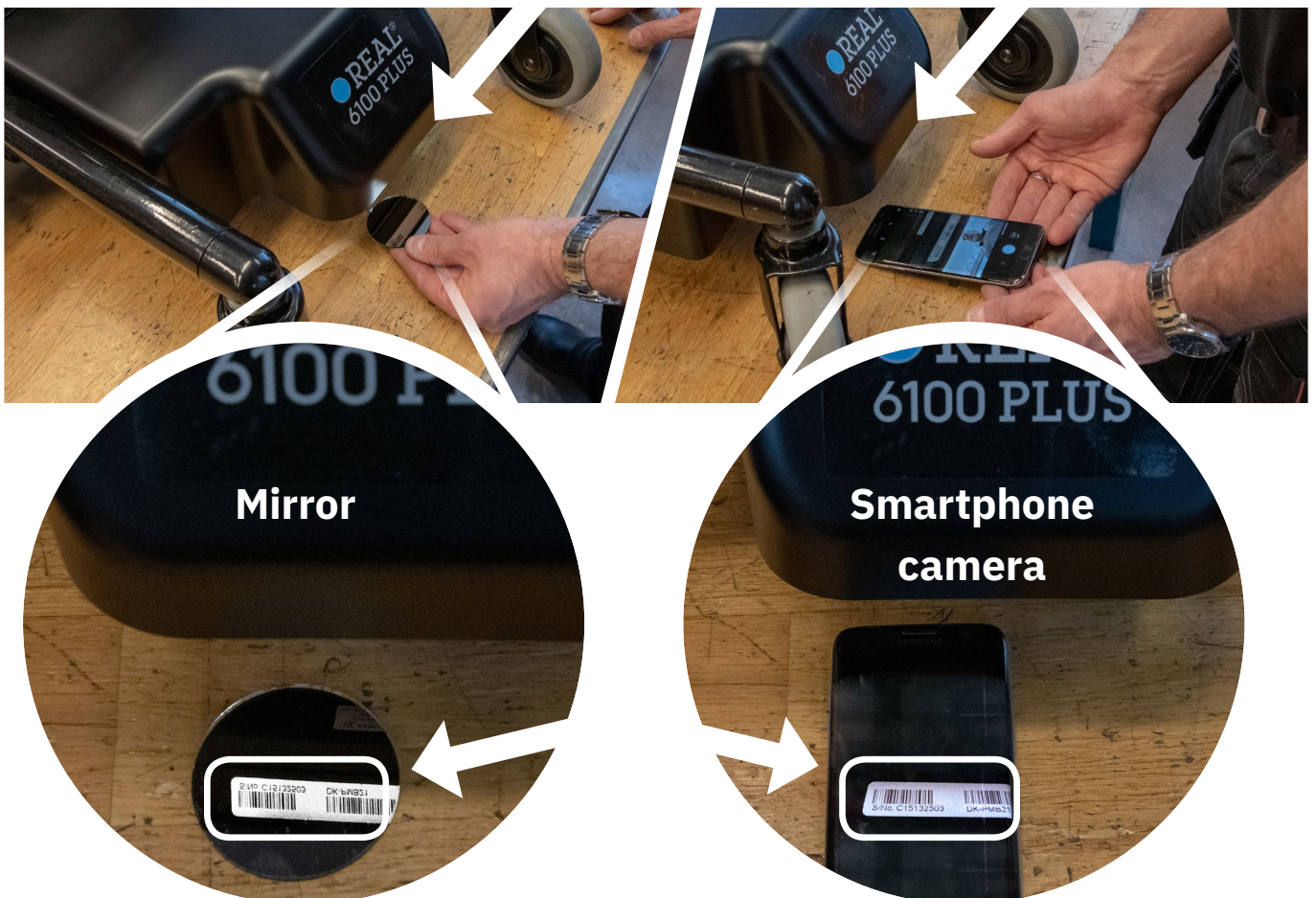
## Del 1: Kontrollera serienummer

Use a mirror or a smartphone camera to easily check the serial number located on the lower edge of the power module. The number is written in the format "S/No. A01234567"

Check the serial number of the power module *Shark* and compare with the list on the next page, to see if the module needs to be replaced. If the serial number is included in the list, continue with part 2 of this document for instructions on changing the power module.

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Be careful if reading the text in the mirror so that it is not misinterpreted

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## List of defective *Shark* power modules

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Dynamic Controls Item Number on all defective  
Shark power modules: DK-PMB21

Dynamic Controls artikelnummer på alla defekta  
Shark Power-moduler: DK-PMB21

S/No.	S/No.	S/No.	S/No.	S/No.	S/No.
B18101513	B18101551	B18183534	D18148190	D18148285	H18199814
B18101514	B18183509	B18183535	D18148191	D18148286	H18199815
B18101515	B18183510	B18183536	D18148192	E18162924	H18199822
B18101517	B18183511	B18183537	D18148193	E18162925	H18199823
B18101523	B18183512	B18183538	D18148196	E18162926	H18199824
B18101524	B18183513	D18141619	D18148197	E18162927	H18199825
B18101525	B18183514	D18141620	D18148198	E18162928	H18199826
B18101526	B18183515	D18141634	D18148200	E18162934	H18199827
B18101527	B18183516	D18141635	D18148266	E18162935	H18199834
B18101534	B18183517	D18148165	D18148276	E18162936	H18199835
B18101535	B18183518	D18148181	D18148277	E18162937	H18199836
B18101536	B18183529	D18148182	D18148278	E18162940	
B18101537	B18183530	D18148183	D18148279	H18199803	
B18101538	B18183531	D18148184	D18148280	H18199804	
B18101549	B18183532	D18148185	D18148281	H18199805	
B18101550	B18183533	D18148189	D18148284	H18199810	

## If defective, order a new power module

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To order a new power module please contact your Mercado Medic representative.

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## If not defective, inform Mercado Medic

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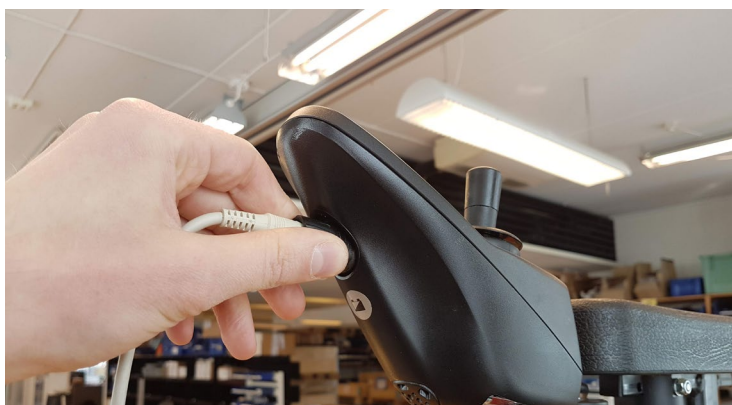
# Part 2: Replace the power module (only if needed)

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If the power module *Shark* has a serial number that belongs to the batch according to the list on the previous page, follow this instruction on how to change the power module.

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### 1.

**Use Wizard to download and save the drive program which is stored in the power module**

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The drive program for the electric wheelchair is stored in the power module. Make sure the program for the specific chair is saved before removing the defective power module. If you have any problem regarding Wizard or saving the program, please contact your Mercado Medic representative.

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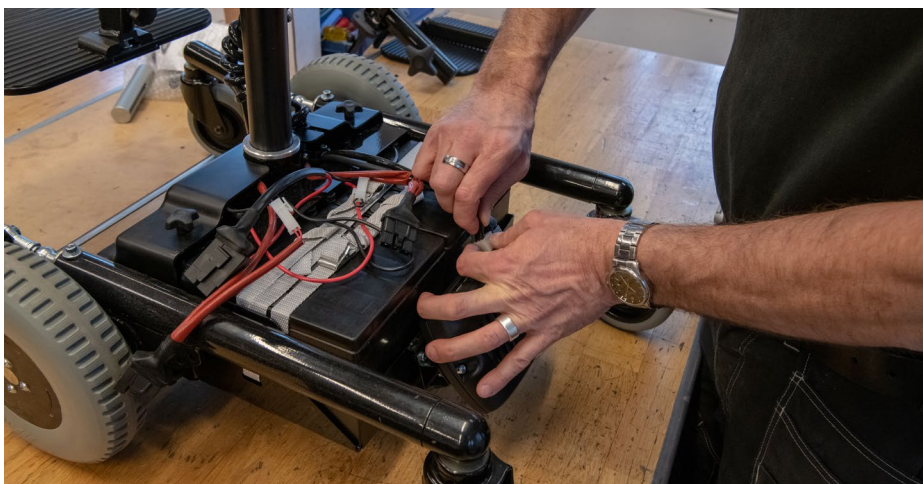
### 2.

**Release the locking knobs (4 pcs) and remove the cover**

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### 3.

**Disconnect the cables from the power module**

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### 4.

**Unscrew the screws on the power module**

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Note that there are two variants on the bracket on which it is mounted. One where the screw attaches with locknuts and one with cage nuts.

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---

## 5.

**Replace the power module. Save the defective one, mark it and send back it back according to step 10.**

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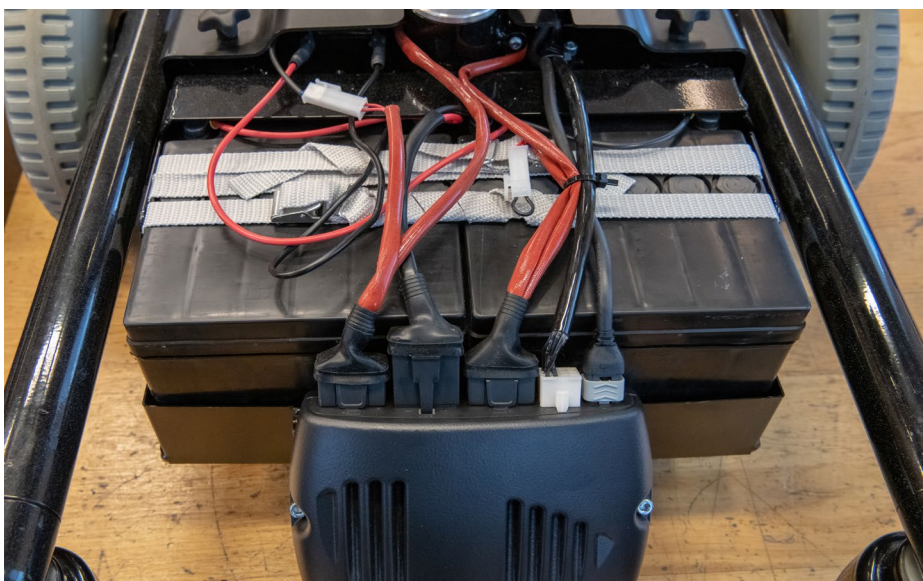
---

## 6.

**Mount the power module and connect the cables.**

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
## 7.

### Mount the cover and tighten the four locking knobs

The rear wheels must be turned outwards so that the cover can be easily mounted

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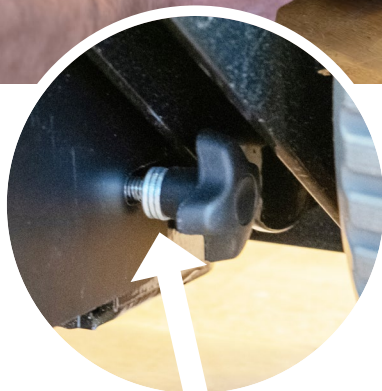Make sure all **three** washers are placed on the outside of the cover before tightening the locking knobs on the sides

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---

## 8.

**Re-program the new power module with Wizard**

Programmera den nya powermodulen med Wizard

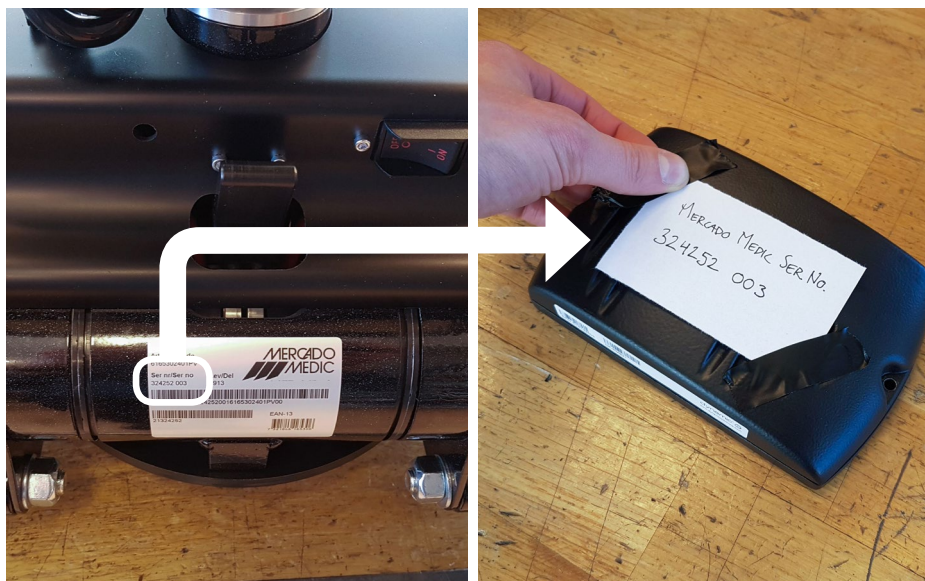
---

## 9.

**Check functionality by testing all electrical functions on the chair**

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## 10.

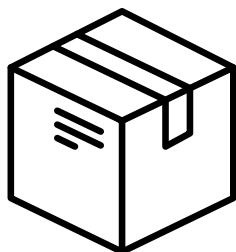
**Send back the defective power module to your Mercado Medic dealer**

**Skicka tillbaka den defekta powermodulen till din Mercado Medic representant**

Mark the power module with the chair's serial number found on the front of the chassi.

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### United Kingdom

Mercado Medic  
Att. Henrik Sellberg  
Repton House, Bretby  
Business Park, Ashby Road  
DE15 0YZ Burton on Trent

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*Region Norr*  
Fredrik Lindberg  
Anbudsgatan 9  
93157 Skellefteå

The English text in the instructions is a translation of the Swedish original. In case of any inconsistency between the Swedish and English version, the Swedish version shall prevail.

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# Appendix A / Bilaga A

## Mercado Medic serial number for chairs that might be affected by the defect

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